U-Tech. Ref F905

Manual de instrucciones
Instruction manual
Mode d'emploi
Bedienungsanleitung
Manuale d'istruzioni

Electroestimulación
Electrostimulation
Electrostimulation
Elektrostimulationsgerät
Elettrostimolazione

Advertencia

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Warning

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

EC-Declaration of compliance

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU
2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU

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Recycling

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

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Welcome!

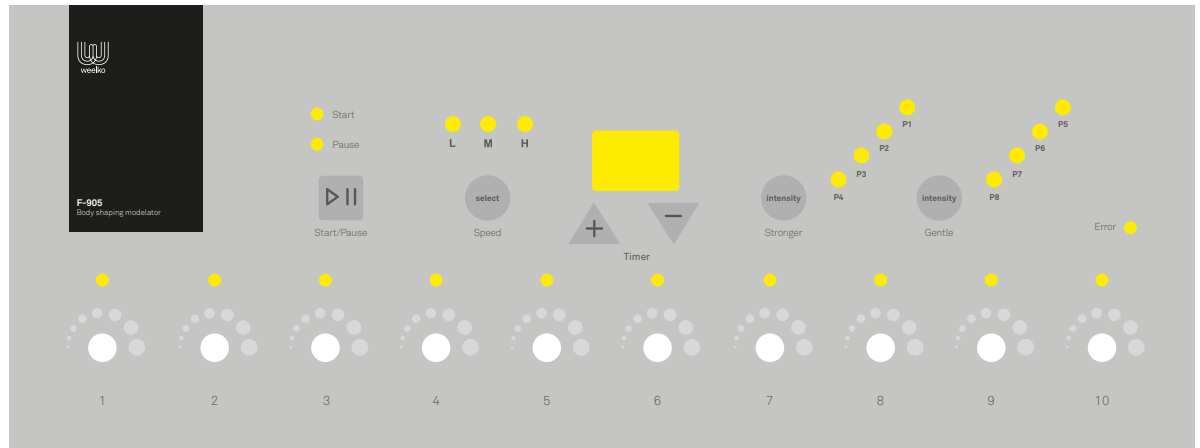
Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

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Instructions of use

Before treatment

1. Rub the skin surface with a damp sponge.
2. Use a vaporizer (or a hot pad) to soften the surface of the skin in the treatment area.
3. After treatment for skin cleansing is recommended to apply some nutritive oil on the skin

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Notes for the placement of electrodes

- Place the electrodes correctly according to the polarity (+ or -). Otherwise the treatment effect will be reduced and the customer will feel uncomfortable.
- Each pair of electrodes should be placed symmetrically according to the schemes shown, the distance between both electrodes should not be less than 2 centimeters.
- Apply conductive gel evenly in the electrode surface.
- As shown in the diagrams shown, use elastic tapes used for a better fixation of the electrode on the skin.
- Check that the electrodes are placed correctly before you start the treatment. Also make sure that the strips are not too tight and the electrodes are in contact with the skin.

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Attaching electrodes and treatments

Programs (INTENSITY STRONGER/INTENSITY GENTLE)

- P1, P5: muscle massage, muscle pain relief.
- P2, P6: Improve muscle tone in chest, waist, abdomen, buttocks, thighs and legs.
- P3, P7: Simulated body exercises to improve blood flow and lymphatic drainage.
- P4, P8: Weight loss program.

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Programme (INTENSITY STRONGER/INTENSITY GENTLE)

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A. Lymph node circulation throughout the body

Time: 25 minutes

Process: select P3 program

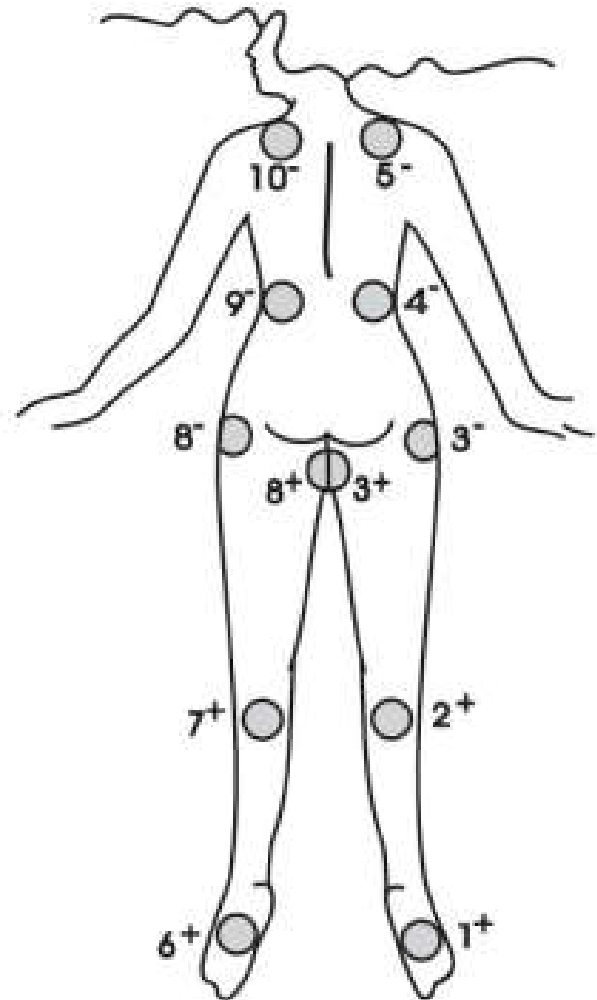
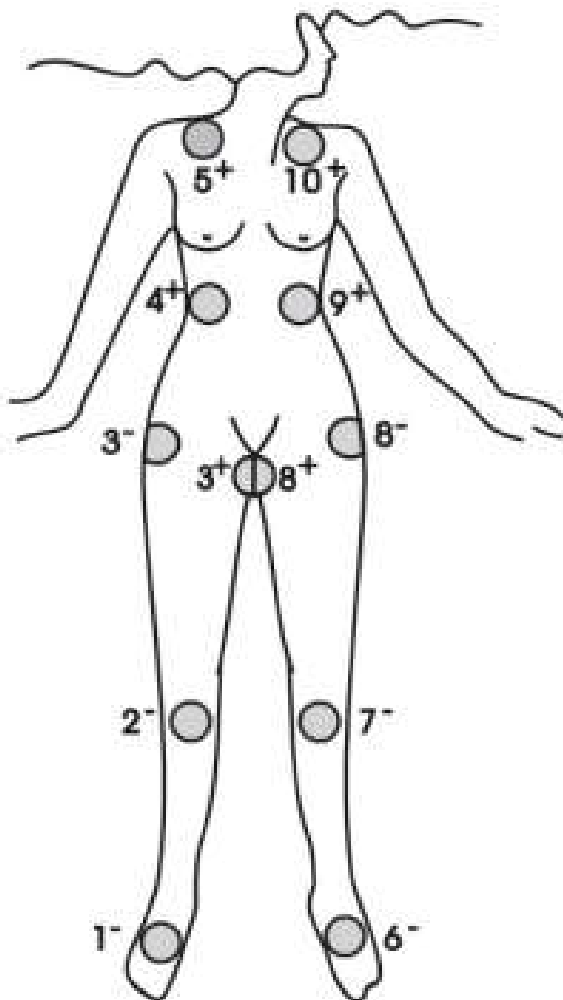
Posture: reclining position (head elevated about 30°)

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B. Lymph node circulation in the lower body part

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N. Back toning

Time: 30 minutes

Electrode arrangement 1: for the upper back part

Electrode arrangement 2: for the lower back part

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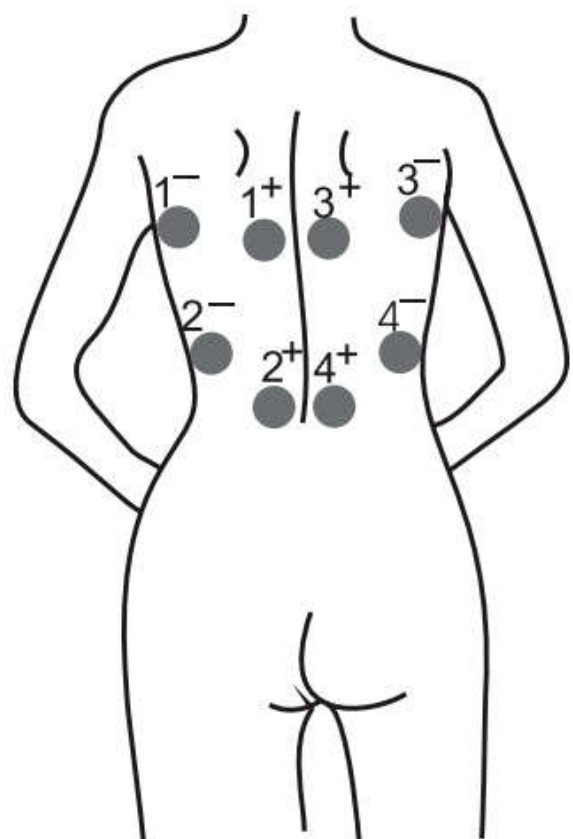
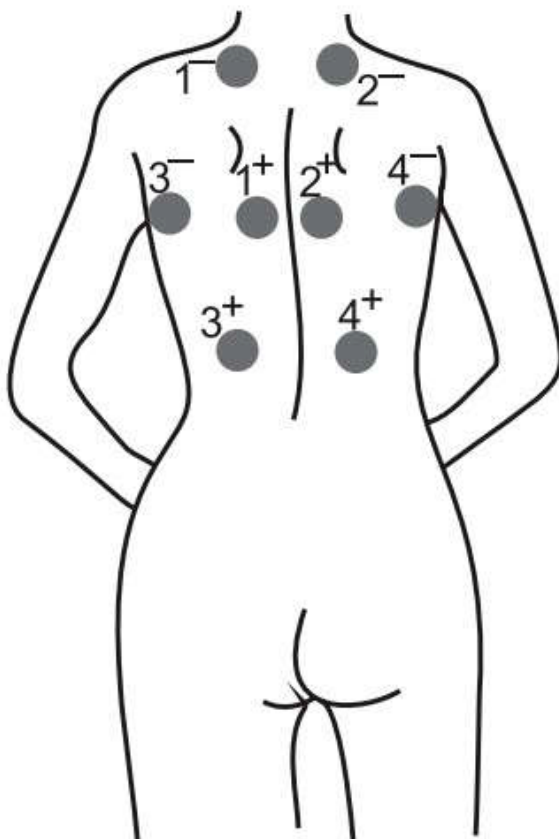
Beginning of the treatment

Make sure that all output regulators are at "MIN" before starting the unit. If the unit turns on when any of the regulators is not set to "MIN" an alarm will sound and the "ERROR" led will flash. At this point you should switch the unit off, adjust the drives to "MIN" and switch on the device. This is a device to prevent unexpected supply shock on power. Anyway, the alarm can be disabled by setting all regulators to "MIN" without turning off the device. In this case, always make sure that no electrode is on the client's skin.

Once you have done this check, the unit can be turned on using the "POWER" button on the rear panel.

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Beginning of the treatment

1. Select the desired output frequency by pressing "SELECT SPEED". You can choose between L (low), M (medium) or H (high).
2. Select the treatment time. The default value is 30 minutes, and can be adjusted using the "TIMER". The maximum working time is 60 minutes adjustable.
3. Select the desired output current using the "INTENSITY STRONGER" or "GENTLE INTENSITY" intending to apply strong or gentle currents respectively.
4. Press the "START/PAUSE" to start the treatment.
5. Adjust, for each of the channels used, the output current through the revolving controller according to the reactions of each muscle.
6. When working time is over, an alarm will alert you. At this moment:
 - a. You can reuse the instrument by pressing "START/PAUSE" (the frequency and intensity settings are the same and the working time is 15 minutes).
 - b. You can complete the treatment setting all regulators to "MIN" and turning off the unit using the "POWER" button on the rear panel.
7. Once the treatment is finished remove the elastic straps and electrodes. Clean the treated area with water or a wet wipe.

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Notes

- In cases where the elastic belt is not long enough, you can use two shorter belts attached by velcro.
 - The programs P1 - P4 emit a strong output current. Apply these programs in non-sensitive areas.
 - Programs P5 - P8 emit a weak output current. Apply these programs in very sensitive areas.
 - Do not place the electrodes in areas close to the heart or on swollen or dilated veins.
 - You may not use this instrument on the following people:
 1. Persons with pacemakers
 2. People with arrhythmia
 3. People with epilepsy
 4. Pregnant women
 5. People with stomach, liver or kidney problems
- Where it is necessary to adjust the output current, it should be increased step by step and in no case it should be done suddenly.

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Caution

- Drink plenty of water and rest during the 10 to 15 minutes after the treatment.
- Do not tighten too strongly the elastic belts , because the client could feel some pinching . On the other hand, this belt should not be too loose for not reducing the effect of treatment.
- In some areas of the body it is not necessary to place the elastic belt to fix the electrodes. In this case is enough with the adhesive layer of the electrode.
- If during the treatment any of the electrodes are releases of skin of the patient, press the "START / STOP" button to stop the machine. Replace the electrode properly before proceeding with the treatment.
- The duration of treatment should never be more than 60 minutes. If it is necessary to repeat the treatment in one area, you must take a break of at least 2 hours.
- If an electrode is placed on a wound in the skin, the patient might feel some discomfort. To avoid this, it is necessary to place the electrode over a bandage.

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