ES

FD2100  
Manual de instrucciones  
WE-2019-FD2100

# VAPORIZADOR OZONO DUPLO

## AVISO IMPORTANTE

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# FACIAL STEAMER DUPLO

## IMPORTANT NOTICE

• Before using the steamer with a customer, it is very important to perform two entire cycles. Do not use it with a patient!

• It is advisable to wash the new cup vessel thoroughly before its first use, with clean water only.

## WARNING

• Do not use any substance other than distilled. Take care when using essential oils or herbs, so as not to contaminate the water used for steaming.

• NEVER point the vapor jet directly down onto the client's face.

• NEVER use the steamer unit in humid or unvented environments.

• NEVER use the steamer outside.

• NEVER attempt to open the steamer casing/housing yourself.

• If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

• Forbid to work without water, if does, it beyond the range of warranty.

## EC-DECLARATION OF COMPLIANCE

This product meets the following requirements:

**1.** Conditions set out in the Low Voltage Directive: 2014/35/EU.

**2.** Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU.

## RECYCLING

 Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

## WELCOME!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.




The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

## PACKAGE CONTENTS

• These components are subject to change without prior notice.

• Appearance of product may differ from the image.

No.	PRODUCT	REF.	QTY
1	FIVE LEG BASE	-	1
2	WHEEL LEG	-	5
3	STRAIGHT POST	-	1
4	SLIDING BAR	-	1
5	TANK SUPPORT	-	1
6	SET SCREW	-	1
7	PRESSING PLATE	-	1
8	FLAT GASKET	-	1
9	GROWER'S SCREW	-	1
10	HEX SCREW	-	1
11	WRENCH	-	1
12	FD2100	-	1
13	WATER TANK	10054	1
14	SEAL	-	1
15	COOL WATER TANK	10007	1
16	WOOL FELT	-	9
17	SCREW	-	1
18	FILLER LID	-	2

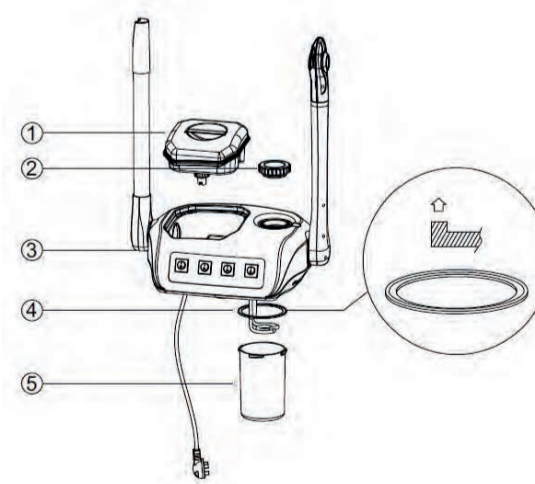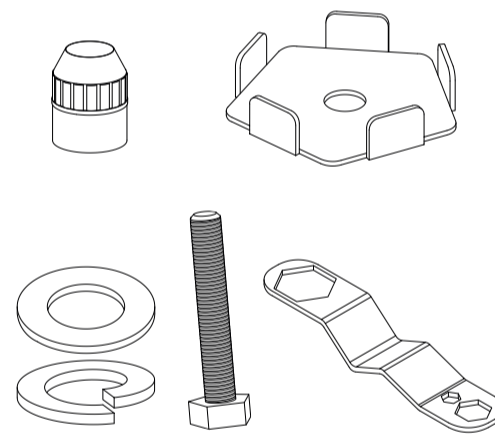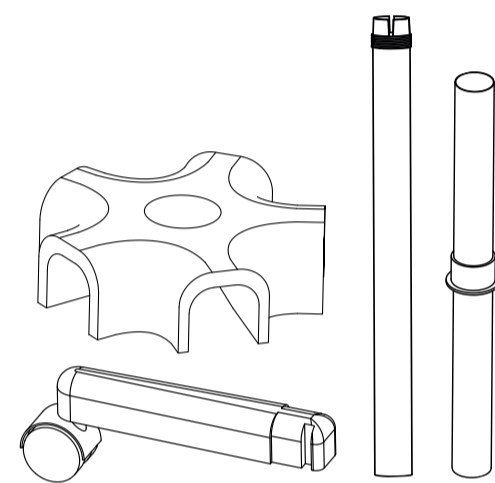
## FRONT PANEL

Moisture Vapor Ozone Power

## INSTRUCTIONS OF USE

### ASSEMBLY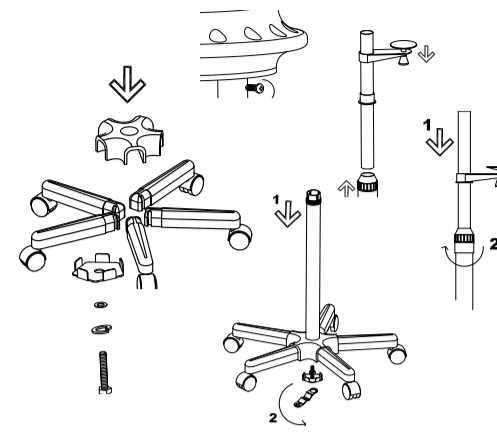


### FILLING WITH WATER

Lift up the water tank of the cool sprayer and pour distilled water into the water cup via the port. Keep the water level between the "MAX" and "MIN" levels.

### FRAGRANCE TREATMENT

Open the blue lid of the upper case of the spray head and pour some drops of essential oil on the felt rag. Close the lid again and start the treatment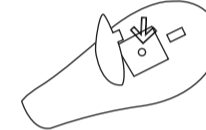

### STARTING TREATMENT


**1.** Turn on the "POWER" button: The heater begins to work. After about 6 min, steam will spray out from the nozzle.

**2.** As of this moment, you can turn on the "OZONE" button: The ozone lamp comes on and starts working.

**3.** Whenever the water level reaches the lower limit, turn off the "POWER" button. Lift up the water tank of the cool sprayer and pour distilled water into the water cup via the port. Keep the water level between the "MAX" and "MIN" levels.

### OPERATION OF THE COOL SPRAYER


**1.** Remove the cold water tank. Unscrew the lid anticlockwise and pour in a metered amount of distilled water.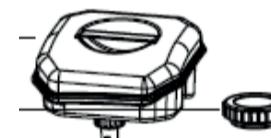


**2.** Place the cold water tank in the correct position and press down.

**3.** Turn on "MOISTURE".

**4.** The water level in the tank will go down to supply the water that disappears from the cistern while the spray is operational. Whenever the water level in the cistern reaches the lower limit and the tank does not contain enough water to supply it, the main computer will cut off the power supply off automatically and stop working. At this time, remove the water tank and pour in a suitable amount of distilled water. Return the tank to the cistern to continue working.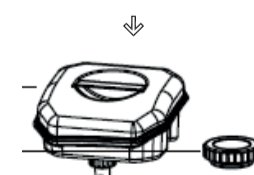

**5.** There is a cap on the bottom to drain tank.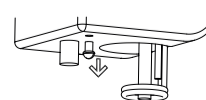

## SUGGESTION OF TREATMENT

SKIN TYPES	DIST.	TIME
Greasy skin, poor blood circulation	25cm	15 min
Normal skin	25cm	10 min
Sensitive skin, dry skin	30cm	5 min
Capillary vessel expansion	30cm	5 min

## CAUTIONS

• When operating the machine, do not point the nozzle at the customer's face until the steam is coming out thinly and evenly.

• The steam should first spray the customer's jaw and then the rest of the face.

## FAQ

If the unit works but does not emit steam.

• Check that the herb lid is properly closed.

• Check that the plastic ring is properly placed between the water cup and the main frame.

• Check that the water cup tray is properly tightened.

• Check the water level.

If the unit emits steam mixed with hot water.

**1.** Clean the water cup with vinegar and rinse with clean water.

**2.** Fill the water cup with clean water and add 8 to 10 drops of vinegar. Turn on the steamer for the time of one operation (do not use it with a patient!).

**3.** When the operation time ends, turn off the steamer and rinse the cup with clean water.

**4.** Use the steamer during the time of 2 whole operations with clean water.

## MAINTENANCE

It is advisable to clean your steamer at least once a week and also to decalcify the water heater element. Please follow the steps below:

**1.** Once the steamer unit is cold, remove the water cup.

**2.** Fill the cup with the appropriate decalcification/ water solution and leave it to stand overnight.

**3.** Remove the water and rinse with clean water to get rid of any residue from the decalcification agent.

**4.** Refill with water and run the steamer through at least one run cycle in order to remove any remaining residue from the steamer unit.


**5.** Remove the cup and rinse as before.

**6.** Refill with water and it is ready for use.

**Note:** Failure to remove all the decalcification residue may cause the steamer to 'spit'. If this occurs, simply repeat the rinse and refill procedure until the issue is resolved.

## ACCESSORIES

REF.	DESCRIPTION
10054	WATER TANK
10007	COOL WATER TANK

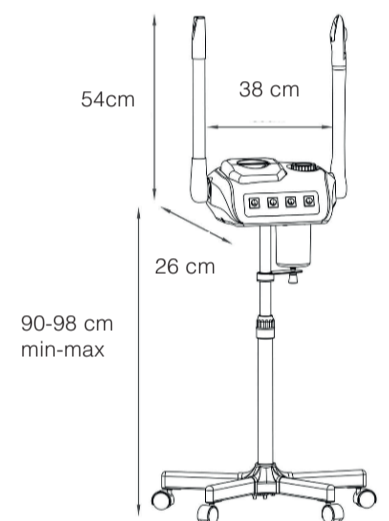## TECHNICAL SHEET

Weights and sizes	Packing size (cm)	86,5x31,5x31
	Gross weight (kg)	8
Water tank	Material	Methacrylate
	Capacity	0,9 l
Working characteristics	Preheat time	6 min. (approx.)
Electrical characteristics	Rated voltage	220V ~ 240V
	Frequency	50Hz/60Hz
	Rated power	850W
Fragrances		Yes
Ozone		Yes
Deposit for herbs		No
Cold spray		Yes

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