B-Equipment. Ref F317A

Manual de instrucciones  
Instruction manual  
Mode d'emploi  
Bedienungsanleitung  
Manuale d'istruzioni  
2021

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## Warning

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

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## EC-Declaration of compliance

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU
2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU

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## Recycling

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

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### **Welcome!**

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

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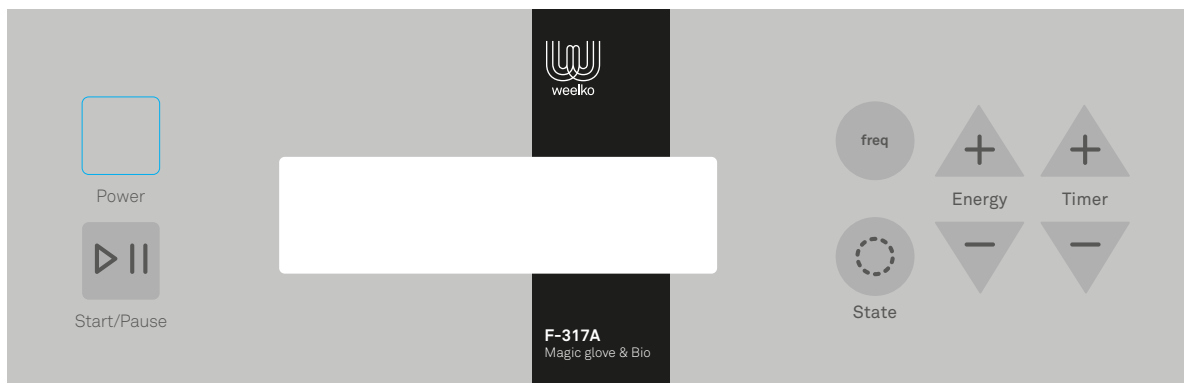
### Instructions of use

1. Connect the power supply via the rear panel.
2. Turn on the "POWER" button.

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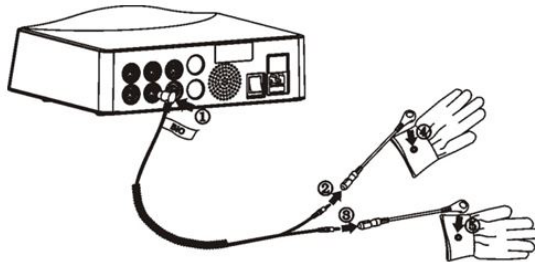
## Magic gloves instructions of use

1. Connect ones side of the wire in the rear panel, in the plug named "BIO WAVE MICROCURRENT".
2. Connect the other two sides of the wires in each glove.
3. The beautician must put isolating gloves under the magic gloves.
4. Make sure that the "ENERGY" level is at the minimum. You can adjust the output level by pressing "ENERGY +/-".
5. Make sure that the "FREQUENCY" level is adjusted according to customer needs. You can adjust the frequency level by pressing "FREQ".
6. Select the desired wave state by pressing "STATE". There are 4 kinds of waves.
7. Select the time of operating by pressing "TIMER +/-".
8. Press "START/PAUSE" to start the treatment.
9. The beautician must massage to beautician with the magic gloves. Adjust the "ENERGY" and "FREQUENCY" level according to the customer feelings. At this moment, the beautician don't have to feel any effect in their hands, and the micro current will not be either noticeable to the customer during this treatment.

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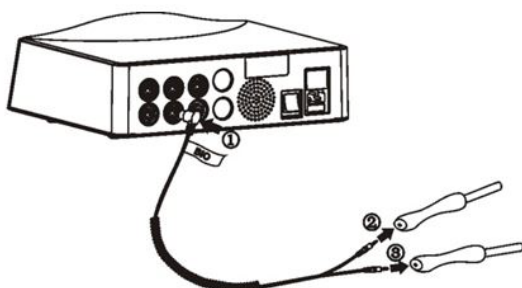
## Bio-lifting instructions of use

1. Connect one side of the wire in the rear panel, in the plug named "BIO WAVE MICROCURRENT".
2. Connect the other two sides of the wires in the handles of BIO-lifting. There are two kinds of handles:
  - The ones that need cotton swabs are used in the eyes part. It is needed to include two cotton swabs in the hole of the handle.
  - The ones with only one pole are used to the full body.
3. Apply enough conductive gel to the treatment area.
4. Make sure that the "ENERGY" level is at the minimum. You can adjust the output level by pressing "ENERGY +/-" button.
5. Make sure that the "FREQUENCY" level is adjusted according to customer needs. You can adjust the frequency level by pressing "STATE".
6. Select the desired wave state by pressing "STATE". There are 4 kinds of waves.
7. Select the time of operating by pressing "TIMER +/-".
8. Press "START/PAUSE" to start the treatment.

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## Notes

- The delivered magic gloves have a length of 20 cm. The isolating gloves must have a length of at least 23 cm. If there are contact between the beautician's skin and the magic gloves, there's no effect with the treatment.
- During the treatment, if the beautician breaks the contact with the customer's skin, the device stops working suddenly. The device restarts when the contact between the magic gloves and the customers skin returns.

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## Cautions

- Pacemaker (any electrical or rhythmic heart-regulating device).
- Pregnancy.
- Thrombosis/Phlebitis
- Skin diseases.
- Operations (within the last 6 months).
- Silicone implants.
- Hypersensitive or vascular skin

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## Maintenance

After each treatment, the magic gloves and the BIO-lifting handles must be cleaned, sterilized and properly saved in a dry place.

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## TECHNICAL SHEET

Weights and sizes	Packing size (cm)	37x27,5x14
	Product size (cm)	21,8x22,5x7,2
	Gross weight (kg)	2,6
	Net weight (kg)	2,1
Working characteristics	Adjustable intensity (10 levels)	60 - 630 $\mu$ A
	Power frequency	2 W
	Working frequency	0,1;5;10;20 30;60 e 120 Hz
Electrical characteristics	Rated voltage	220V ~ 240V
	Frecuency	50Hz/60Hz
	Rated power	4 W

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