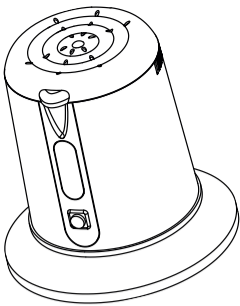
ES - EN

S01D
Manual de instrucciones
Instruction Manual
WE 2018-S01D

ESTERILIZADOR DIGITAL DIGITAL STERILIZER

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ENGLISH

WARNING

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.

- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms, etc. to prevent damage to the electrical components of your unit.

- Never use the unit outdoors; exposure to the elements can cause damage.

- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

- Do not use if the glass bead has been broken, since broken beads are dirty and hard to clean.

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environmental protection in regard to the treatment of electrical waste.

WELCOME!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.


The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

PACKAGE CONTENTS

- These components are subject to change without prior notice.

- Appearance of product may differ from the image.

No.	PRODUCT	REF.	QTY.
1	S01D	-	1
2	QUARTZ BALL BAG	40003	1

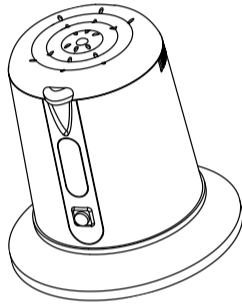INSTRUCTIONS OF USE

1. Fill the sterilizer with all the quartz balls supplied and close the lid.

2. Turn on the unit by pressing the "POWER" button.

3. Press the switch once: it displays "15". It will take 15 minutes of heating to reach the required sterilizing temperature. During the warm-up period, the display shows "15" and the decimal point on the right flashes.

4. After 15 minutes of heating, the buzzer sounds three times to remind you that the machine is ready for sterilizing and the display turns to "00" and enters standby.

TECHNICAL SHEET

Weights and sizes	Packing size (cm)	22x22x22
	Gross weight (kg)	1,25
	Product size (cm)	20x20x19
<hr/>		
Working characteristics	Capacity	152 ml
	Temperature	160-180 °C
	Preset working time	15 min
<hr/>		
Electrical characteristics	Rated voltage	220V ~ 240V
	Frecuency	50Hz/60Hz
	Rated power	80 W

5. You can then insert the tools and press the switch. The lid must be open. The display shows "20" seconds and counts down while sterilizing. After 20 seconds, the buzzer sounds three times to remind you that sterilization is complete. Do not forget to remove the tools. Next, the machine turns automatically to "00" again and enters "standby" status. Repeat the steps above for further sterilizations.

NOTES

- The machine is designed with a self-protecting function which activates 8 hours after powering on. If the machine is in standby status at that time, the buzzer will sound 5 times and automatically stop heating.

- After the machine has entered self-protecting status, if you want to use it for sterilizing again, you have to switch off the power for a second then restart, but it will take another 15 minutes to heat it up for sterilizing again.

CAUTIONS

- To process the high-temperature sterilization of the tools, it is necessary to clean the machine in advance. Wipe it with a cloth before inserting them after the sterilizers.

- While the machine is operational, avoid covering it with cotton towels, plastic bags, etc.

- Do not use any tools in this machine that cannot stand high temperatures, such as plastic spatulas or mask sticks.

MAINTENANCE

- The quartz balls and inner housing need to be cleaned after everyday use.

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© 2018 Weelko