ES

WKB002  
Manual de instrucciones  
WE-2020-WKB002-1

# PRESOTERAPIA Y TERMOTERAPIA

## ADVERTENCIA

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# PRESSOTHERAPY & THERMOTHERAPY



incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

## PACKAGE CONTENTS

- These components are subject to change without prior notice.
- Appearance of product may differ from the image.

No.	PRODUCT	QTY.
1	RIGHT LEG	1
2	LEFT LEG	1
3	RIGHT ARM	1
4	LEFT ARM	1
5	RIGHT FOOT	1
6	LEFT FOOT	1
7	WAIST	1
8	CABLE BLANKET	1
9	TUBES	1
10	MAIN UNIT	-
11	FEEDER CABLE	-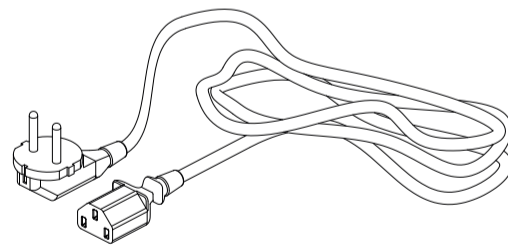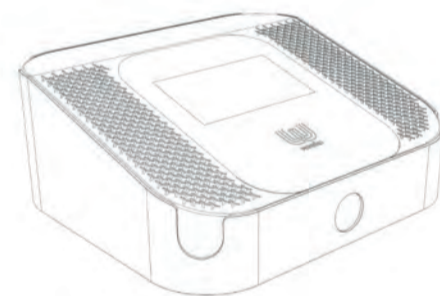

## INSTRUCTIONS OF USE

1. Connect the power cord via the rear panel.
2. Connect the tubes properly to the rear panel. Follow the numerical instructions. Thinner cables from 1 to 5 are used for thermotherapy and thicker cables from 0 to 9 are used for pressotherapy.
3. Place the blanket with the cables on a flat surface so that the patient can lay down on it. Cover the blanket with disposable paper to ensure the best hygienic conditions.
4. Wrap the suit around the user's body. Remember to use a disposable garment underneath the pressotherapy suit.
5. Turn on the power button on the front panel and click on Weelko logo to select the desired menus.
6. Select "pressotherapy" for pressotherapy treatments or "far infrared" for thermotherapy treatments.


If you don't want to use a concrete mode, the device allows to choose sections separately and to adjust the inflation intensity of each section. You just need to press the area you want to treat and adjust its intensity by using the buttons "+" and "-".

8. The "LEVEL" menu allows to adjust 3 inflation intensity levels.

9. The "TIME" menu allows to adjust the treatment time. The default duration is 30 minutes.

10. In the thermotherapy function, the device is equipped with 5 treatment sections. Temperature can be adjusted separately in each section. The "TIME" menu is used to adjust the treatment time. The default duration is 30 minutes.

11. Press "PLAY/PAUSE" to begin or pause the treatment.

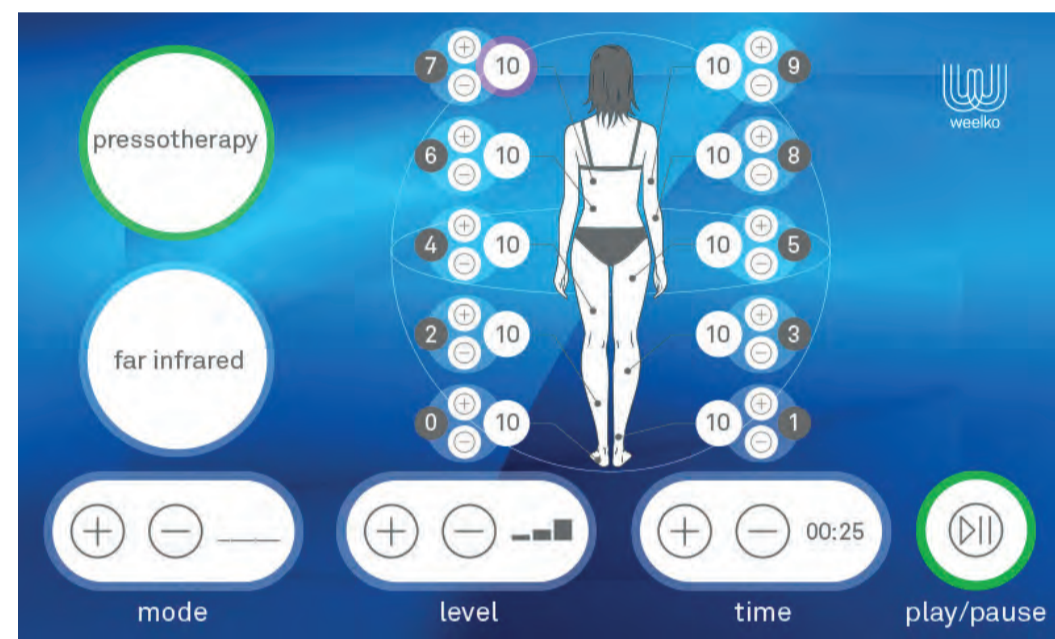7. In the pressotherapy function, the "MODE" menu allows you to choose between four programs. It is recommended to start with C and to select the other programs depending on patient's needs.

• Mode A fills with air sections from 0 to 9. Section 0 is deflated when section 1 is inflated. Section 1 is deflated when section 2 is inflated and so on from one section to another. At the end of the program, sections from 0 to 9 are inflated at the same time and the program starts again.

• Mode B fills with air sections from 0 to 9. Section 0 is deflated when section 2 is inflated. Section 1 is deflated when section 3 is inflated and so on from one side to another. At the end of the program, sections from 0 to 9 are inflated at the same time and the program starts again.

• Mode C fills with air sections from 0 to 9. First section 0 is inflated, then section 1, later section 2 and so on through all sections. At the end, the program starts again from section 0.

• Mode D fills with air the three sections at the same time: legs, abdomen and arms from bottom to top.

## MAINTENANCE

• Connect and disconnect the tubes carefully to extend the product lifetime. However, this device is supplied with a blanket to store the cables so that they are all connected and disconnected at the same time. Only the general power cord at the rear part of the device should be connected and disconnected.

• After use, clean the suit with a wet cloth.

• After a thermotherapy treatment, open the suit to remove stored heat during the treatment.

## CAUTIONS

• Be careful not to wrap the airbag too tight, to avoid hurting the users when the air is too strong.

• If the surface of the blanket is broken or aging, please replace it with a new blanket to prevent accidental skin burns or electric shocks.

• During the treatment, check with the patient if the temperature is appropriate.

• It is recommended for the patient to drink plenty of water before and after treatment to avoid dehydration and heatstroke.

• Use a cellulose protector underneath the pressotherapy suit to ensure the best hygienic conditions.

• Never leave the suit heating with nobody inside.

## ACCESSORIES

RIGHT LEG
LEFT LEG
RIGHT ARM
LEFT ARM
WAIST
RIGHT FOOT
LEFT FOOT
TUBES

## TECHNICAL SHEET

Weights and sizes	Packing size (cm)	66x45x40
	Gross weight (kg)	18,5
	Product size (cm)	34x38x15
Working characteristics	Working power	350 W
	Air pressure	0-0,4 kg/cm <sup>2</sup>
	Air flow	0,40 L/min.
	Rhythm of air pressure	0-15 s.
	Air channels	10 (20 bags )
Electrical characteristics	Temperature range	30-50°C
	Rated voltage	220V ~ 240V
	Frequency	50Hz/60Hz
	Rated power	350 W

## WARNING

• Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.

• Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms, etc. to prevent damage to the electrical components of your unit.

• Never use the unit outdoors; exposure to the elements can cause damage.

• If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

• Do not use on pregnant women, on people with pace-makers (any electrical or rhythmic heart-regulating device) or with skin infections.

## EC-DECLARATION OF COMPLIANCE

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU

2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU

## RECYCLING

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

## WELCOME!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to

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© 2020 Weelko