Hightech Ref. WKB005

Manual de instrucciones

Hightech Combi Hydro

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Ultrasonidos

Martillo Frío

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Spray

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Advertencia

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Skin Scrubber

Warning

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

Recycling

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

EC-Declaration of compliance

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU
2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU

Welcome!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

weelko
the wellness
company

Importador/Importer
Importateur/Importeur
Importatore:

Weelko Barcelona, S.L.
ESB65397556

Barcelona
Pol. Ind. La Valldan
Camí de Garreta, 92
08600 Berga
Barcelona – Spain
T. +34 938 24 90 20
E. info@weelko.com
W. www.weelko.com

Package contents

Contraindications

Do not use the device with:

- Patients with hypersensitive skin or severe acne.
- Patients with diabetes, high blood pressure or high cholesterol.
- Patients with contagious skin infections.
- Patients with pacemakers, patients with implantable cardioverter-defibrillator (ICD) or any other electrical or rhythmic heart-regulating device.

waste bottle. If the indicator light flashes, empty the bottles and place them correctly again.

Fill the bottles with the following products:

- A: Deep cleaning liquid/agent
- B: Moisturizing liquid
- C: Disinfectant solution
- D: Oxygenic liquid (H2 O2 rear bottle)

Start-up



- Open the box and check that all parts are included and in good condition. Please keep the original package for possible service instances.
- Check if the device is in good condition. If there's any problem, please contact your distributor.
- Locate the device in a clean and well-ventilated area with a temperature between 5°C and 40°C. The relative humidity cannot be greater than 80%. Make sure no objects are located within a distance of 5 cm around the device.
- Do not place any objects on top of the device.
- Place front and rear bottles correctly. To extract the rear ones, unscrew them. For the front ones, just press the upper button and they will be easily removed.
- Plug the power cord into the socket and switch on the device.
- Clean the bottles before filling them with beauty products. When extracting the bottles for emptying and cleaning after a treatment, make sure the device is not in operation, otherwise the beauty product could enter the housing of the device and may cause damage to its components.
- If you hear a buzzing sound when the device is in operation, please check the monitor indicator of the

HIDRODERMABRASION

Indicated for deep cleaning and for treatments to remove black spots, acne, dirt from skin and for oily skins. It is also indicated together with fat reduction treatments and to increase the skin's elasticity and tone.

1. Plug the cable into the "Abrasion" side socket. Place it so that "Open" remains on the upper part of the cable.
2. Clean the patient's face before treatment.
3. Depending on the area to be treated, insert the corresponding head into the handle.
4. Depending on the product to be used, turn central wheel to choose the bottle. For example, if an oxygenic treatment is to be applied, select letter D.
5. In between treatments with different products, press and hold for 4-5 seconds the head of the handle twice with your finger to clean it.
6. Change the head and adjust the suction intensity depending on the patient's skin and the area to treat.
7. Press the "H2 O2" or "Dermabrasion" button on the screen depending on the treatment to perform. Wait 3 minutes before starting to spray. This function can

Display and buttons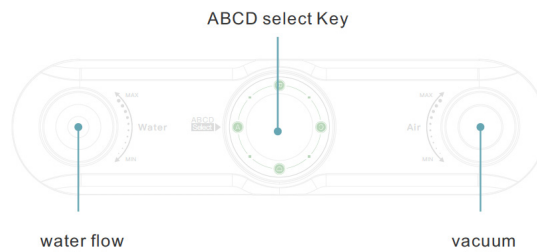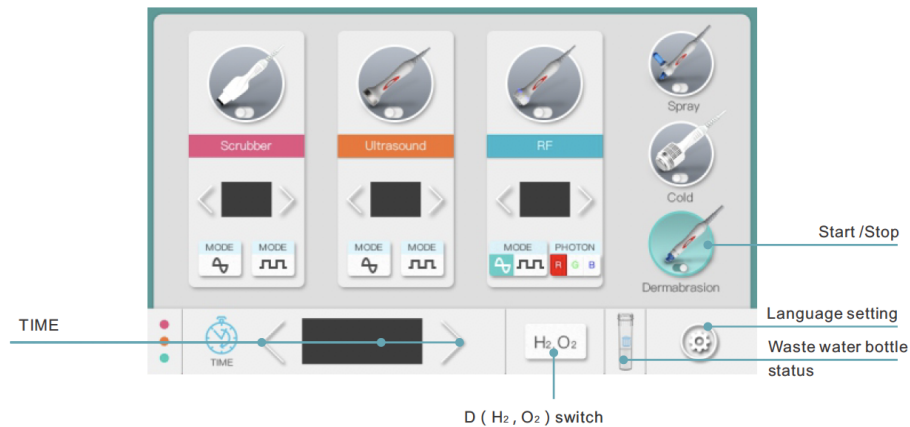

also be selected from the handle's button.

8. After treatment, clean the bottles ABCD and tubes to avoid blockage.

ULTRASOUND

1. Plug the cable into the "Ultrasound" side socket. Place it so that "Open" remains on the upper part of the cable.
2. Clean the patient's face before treatment and apply some gel on the area to be treated.
3. Select treatment time, adjust intensity from 1 to 10 with the arrows and select the type of wave using the "MODE" buttons: continuous wave (left button) and pulsed wave (right button). It is recommended to use the first one for treatments on normal skins and thicker tissue areas. The second one, however, is ideal for sensitive skins and soft tissues. It is recommended to start with the continuous wave and to gradually increase the intensity.
4. Press the "Ultrasound" button on the screen to start the treatment.
5. Move the handle by gently pressing the skin from bottom to top along the contour lines during 5-8 minutes.

COLD HAMMER

Indicated after skin rejuvenation treatments, to diminish the size of pores, to remove redness, to decrease skin sensation and to improve skin's tone.

1. Plug the cable into the "Cold" side socket. Place it so that "Open" remains on the upper part of the cable.
2. Clean the patient's face before treatment and apply some gel on the area to be treated.
3. Press the "Cold" button on the screen.
4. Place the hammer on the patient's areas to be treated.

RADIOFREQUENCY

1. Plug the cable into the "RF" side socket. Place it so that "Open" remains on the upper part of the cable.
2. Clean the patient's face before treatment and apply some gel on the area to be treated.
3. Select treatment time, adjust intensity from 1 to 10 with the arrows and select the type of wave using the "MODE" buttons: continuous wave (left button) and pulsed wave (right button). It is recommended to use the first one for treatments on normal skins and thicker tissues. The second one, however, is ideal for sensitive skins and soft tissues. It is recommended to start with the continuous wave and to gradually increase the intensity.
4. Press the "RF" button on the screen to start the treatment.
5. Move the handle by gently pressing the skin from bottom to top or make small circles on the entire surface of the face.
6. The operation should not last more than 10-15 minutes.

SPRAY


1. Plug the cable into the "Spray" side socket. Place it so that "Open" remains on the upper part of the cable.
2. Clean the patient's face before treatment.
3. Fill the lotion to spray into the small plastic container which is adapted to the spray pipe .
4. Press the "Spray" button on the screen.
5. Point the pipe towards the area to treat and start spraying the lotion.
6. If at some point the nozzle is blocked up, clean away any accumulated waste using a needle or a small tool.

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Eliminates dead skin cells, removes the excess of cuticles and improves skin texture.

MODE A: Scrubber

1. Plug the cable into the "Scrubber" side socket. Place it so that "Open" remains on the upper part of the cable.
2. Clean and dry well the patient's face before treatment. Apply some exfoliating product on the area to be treated.
3. Select treatment time, adjust intensity from 1 to 10 with the arrows and select the type of wave using the "MODE" buttons: continuous wave (left button) and pulsed wave (right button). It is recommended to use the first one for treatments on normal skins and thicker tissue areas. The second one, however, is ideal for sensitive skins and soft tissues. It is recommended to start with the continuous wave and to gradually increase the intensity.
4. Press and hold down simultaneously the two side buttons of the scrubber probe and tilt it 45° as shown on the image below.
5. Treatment time is around 15-20 minutes.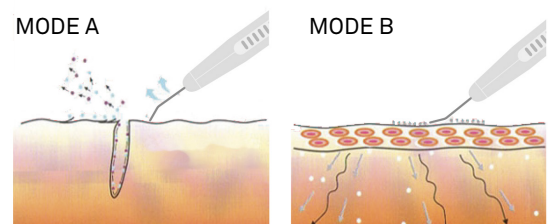

MODE B: Galvanic

1. Plug the cable into the "Scrubber" side socket. Place it so that "Open" remains on the upper part of the cable.
2. Clean and dry well the patient's face before treatment. Apply the proper lotion for the galvanic function on the area to be treated.
3. Select treatment time, adjust intensity from 1 to 10 with the arrows and select the type of wave using the "MODE" buttons: continuous wave (left button) and pulsed wave (right button). It is recommended to start with the continuous wave and to gradually increase the intensity.
4. Press and hold down simultaneously the two side buttons of the scrubber probe, press it firmly on the skin and glide it smoothly over facial lines, as shown in the picture above.

CAUTIONS

- Within 24 hours of receiving the treatment, it is recommended to use sun protection cream. Consult your beautician about which other cosmetic products can be applied on your skin.
- Patients suffering from anaphylaxis, severe acne, hyper-sensitive skin or blood circulation problems should consult a doctor before using this device.
- If the patient notices some discomfort during the treatment, stop it immediately.

MAINTENANCE

- Do not clean the device when it is turned on.
- Use a clean, dry cloth to clean the device. You can use a slightly damp cloth to clean off stubborn dirt or stains.
- To clean the hydrodermabrasion handle, start with bottle C and select the maximum suction intensity. Press "Start" and keep a finger on the head for 5-10 seconds. Remove the head and repeat the movement 8-10 times. When bottle C is clean and empty, fill bottles A and B with warm water and repeat the same action to prevent tubes and handles blockage. Afterwards, extract the head and clean it with a cloth to remove any kind of dirt. You can also soak the heads in alcohol or disinfectant solution for a minute.
- To clean the spray pipe, fill the pipe container with warm water, press "Start" and spray all the warm water contained inside.
- Use a damp cloth to clean the other probes and handles after each use.

TECHNICAL SHEET

Weights and sizes	Packing size (cm)	43x45,5x31
	Gross weight (kg)	9,5
	Product size (cm)	26x40x38
	Screen size (inches)	7"
Working characteristics	Ultrasound Freq.	1 MHz
	Bipolar RF Freq.	1 MHz
	Suction power	650 cm/hg
	Water flow power	30 l / h
	Vacuum pressure	80 Kpa
Electrical characteristics	Rated voltage	230 V
	Frequency	50Hz/60Hz
	Rated power	100 W

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