ES - EN

W-003  
Manual de instrucciones  
Instruction Manual  
WE 2013-W003-1

# CALENTADOR DE CERA (3 CARTUCHOS) WAX WARMER (3 TANKS)

## ESPAÑOL

### ADVERTENCIA

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### WELCOME!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

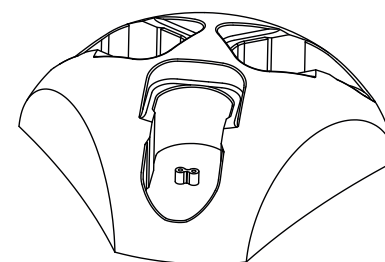
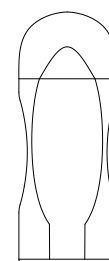
The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

### PACKAGE CONTENTS

• These components are subject to change without prior notice.

• Appearance of product may differ from the image.

No.	PRODUCT	REF.	QTY.
1	APPLICATOR	-	1
2	BASE	-	1

### INSTRUCTIONS OF USE

**1.** Place the depilation wax in the machine.

**2.** Switch on the unit. After 20 minutes, the wax can be taken out and rolled onto the depilation paper for use.

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## ENGLISH

### WARNING

• Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.

• Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.

• Never use the unit outdoors; exposure to the elements can cause damage.

• Don't connect the power when there is no wax in the wax machine.

• If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

### EC-DECLARATION OF COMPLIANCE

This product meets the following requirements:

**1.** Conditions set out in the Low Voltage Directive: 2006/95/EC

**2.** Conditions set out in the Electromagnetic Compatibility Directive: 2004/108/EC

### RECYCLING

 Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

### TECHNICAL SHEET

Weights and sizes	Packing size (cm)	16x8,5x20
	Gross weight (kg)	0,7
	Product size (cm)	19x4x7
Electrical characteristics	Rated voltage	220V ~ 240V
	Frecuency	50Hz/60Hz
	Rated power	10 W

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