Move. Ref P-01

Manual de instrucciones
Instruction manual
Mode d'emploi
Bedienungsanleitung

Ultrasonidos portátil
Ultrasound portable
Ultrasons Portable
Tragbares Massagegerät
Ultraschall

Advertencia

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Warning

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

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EC-Declaration of compliance

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2006/95/EC
2. Conditions set out in the Electromagnetic Compatibility Directive: 2004/108/EC

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Recycling

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

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Welcome!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

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Instructions of use

Before treatment

1. Cleanse/tone the treatment area using the prescribed skin-type products.
2. Remove any left-over product by facial steam for 10 minutes.
3. Cleanse the treatment area again to remove any dirt or debris from the face in order to prevent the ultrasound waves from pushing it back into the dermis.
4. Tone every part of the face (or treatment area) gently using a suitable astringent.
5. After wiping off the astringent, the skin will be thoroughly cleansed for optimum protection.
6. Apply enough conductive gel to the treatment area.

Starting treatment

1. Press the "ON/OFF" button.
2. Select the desired output level by pressing "UP/DOWN".

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Cautions

- Never use on dry skin.
- Do not use on the stomach area of pregnant women.
- Do not use on people with pace-makers (any electrical or rhythmic heart-regulating device).
- Do not use on people with known heart conditions.
- Always use ultrasonic electrodes with enough moisturizer or serum on the skin
- Ultrasonic treatment is suitable for all skin types. The massage time and machine settings can be adjusted for each different skin type. For sensitive skins, reduce treatment time to 3-5 minutes and make sure the power setting is at the lowest level to start with.

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FAQ

Check the following if the work unit switches on but the probes do not emit.

- Put a drop of water on one probe.
- Switch on the unit and slowly increase the output energy.
- If the water starts to move and evaporate, the unit and the probes are working properly. If not, the probes are not working properly.

The unit and the probes are working properly but the customer doesn't feel any effects.

- Make sure enough conductive gel has been applied to the treatment area.

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Maintenance

- The ultrasound probes should be cleaned using clean water (not hot) and a clean, soft cloth and then chemically sterilized using an appropriate sterilization fluid. Dry the probe carefully after sterilizing and store it safely in its holder on the trolley in between uses. Always sterilize the probes again prior to use.
- Never immerse the probe in any fluid!

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