ES Manual de instrucciones
EN Instruction manual
FR Mode d'emploi
DE Bedienungsanleitung
IT Manuale d'istruzioni

WKV001

SKY

VAPORIZADOR OZONO
OZONE FACIAL STEAMER
VAPORISATEUR À OZONE
BEDAMPFER MIT OZON
VAPORIZZATORE OZONO

USO DEL MANUAL

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USE OF THE MANUAL

It is important that you read and retain this user manual for future reference. It contains important information on its use and maintenance.

You can also ask a professional for guidance to make sure you are using the bed correctly.

WARNING

- Never under any circumstances attempt to open or inspect the internal components of the device. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the device in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards or contact the dealer for service.

EC-DECLARATION OF COMPLIANCE

This product meets the following directives:

- Low Voltage Directive: 2014/35/EU
- Electromagnetic Compatibility Directive: 2014/30/EU

RECYCLING

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve natural resources and improve standards of environmental protection regarding the treatment of electrical waste.

WELKOME!

Thank you very much for your purchase. Please read carefully the instructions before using the device. We accept no responsibility for damages caused by misuse.

The company reserves the right to amend the details of the devices with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

IMPORTANT NOTICE

- Before using the steamer with a customer, it is very important to perform two entire cycles. Do not use it with a patient!
- It is advisable to wash the new cup vessel thoroughly before its first use, with clean water only.

OPERATION PRINCIPLE

The electric resistance heats the water inside the tank and make it boil. The steam comes out from the nozzle at the top of the steamer arm. The ozone is an antiseptic, bactericidal, disinfecting and oxygenating gas, so turn on the OZONE button in the machine to start the treatment with all these benefits.

PRODUCT ASSEMBLY

- Insert the white axis into the base and fix it with the provided screws. Tighten the nut with the key as hard as to assure it becomes properly fixed.

EN

- Introduce the metallic axis into the white axis. There is a lifting spring inside. Adjust the height and tighten the black part that joins both rods (white and metallic ones).

- Insert the main unit of the machine inside the superior metallic bar.
- Introduce the water tank into its place by ensuring the rubber joint is well placed and screw the fixation mechanism that holds the tank on the inferior part.

OPERATION

- A. Fill the water cup with demineralised or distilled water until the limit line. Do not pour essence oils or aromatic herbs inside the water tank.
- B. Connect the machine to electricity and turn on the power switch.
- C. After about 6-8 minutes, water steam begins to come out from the spray nozzle.
- D. Press the OZONE switch and ionized steam will be coming out.
- E. Before pointing the steam towards the patient's face, please check that the steam is coming out properly.
- F. The security distance to be respected between the patient and the steamer arm is 30-35 cm.
- G. The temperature of the steam reaches between 35 and 40 °C.
- H. The machine is equipped with a timer. You can program the duration and the machine will automatically stop once the time is over.

EFFECTS

- Thanks to the ozone function: bactericidal, disinfecting and oxygenating.
- Thanks to the heat: moisturising and vasodilator.
- Relaxing action
- Heat dilates the pores size which helps extracting blackheads.

CONTRAINDICATION

- It is not recommended for skins sensitive to heat, to red patches or to congestion. The ozone gas is a bit irritating for the sensitive skin. The spraying is not recommended either for asthmatic or claustrophobic people.

WARNING

- Before using the steamer with a patient, please make two complete circuits in order to check the operation. It is also recommended to make a couple of circuits more with some drops of vinegar into the water in order to clean all the internal tubes.
- You should also clean the water tank before using it for the first time.
- Never point the nozzle directly towards the patient's face without respecting the security distance. Let the steam come out before using it with the patient to assure it comes out correctly.
- Do not use the machine in wet or non-ventilated places.
- Do not use the steamer outdoor.
- Do not try to open the main unit without the supplier help or instructions.
- If the power cord is damaged, please ask the supplier to give you a new one. Do not use the machine.
- Do not leave the machine working without water. This will extinguish the guarantee.
- Do not pour chemical products or oils inside the water tank. This can cause the steamer to pour water instead of steam.
- After operation, empty the water tank and clean it. Once per week the tank should be cleaned with vinegar.
- Adjust the time of the treatment according to the patient's skin.

MAINTENANCE

- The lime can gather inside the machine according to the hardness of the water in some regions. So, attention, always use demineralised or distilled water.
- In any case you also need to regularly maintain the machine as the lime accumulation can lead to a malfunction of the machine, so please fill the tank with water and some vinegar and make a couple of complete cleaning circuits to clean the inside.

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